

# CUTIMED® SILTEC® SORBACT® Microbe-Binding Silicone Foam Dressing

FOR ENHANCED FLUID AND INFECTION MANAGEMENT

Helps maintain a moist wound environment for improved wound healing

Gentle fixation silicone border

**Siltec® Foam Technology**  
**+ Sorbact® Antimicrobial Technology**



Protects wound edges through vertical absorption with highly absorbing open-porous polyurethane foam

Sorbact® Wound Contact Layer helps manage and prevent infections

Increased absorption capacity due to super-absorbent hot-melt stripes

# ABSORPTION THAT ADAPTS + SORBACT® TECHNOLOGY


Cutimed® Siltec® Sorbact® is an innovative combination of reliable absorption and Sorbact® Technology. The super-absorbent particles within the foam layer effectively adjust to absorb moderate to high, exudate and retain it even under compression.

## FLUID HANDLING CAPABILITY

- Super-absorbent hot-melt stripes ensure vertical absorption of exudate reducing the risk of maceration.
- Dynamic MVTR allows fluid to transpire while also maintaining a moist wound environment.
- Good flexibility and conformability with new unique sizes. Easy application and reliable adhesion on contoured body parts and joints.
- Silicone layer minimizes pain with atraumatic dressing changes, and wound bed remains undisturbed. This may help patient comfort and compliance.




## RELIABLE ABSORPTION – EVEN UNDER PRESSURE

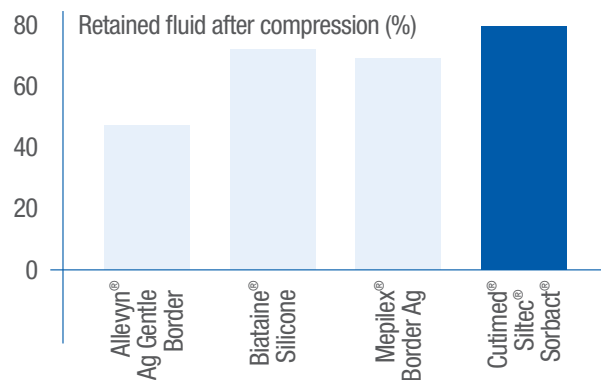

Lab test: Cutimed® Siltec Sorbact® has the highest retention under compression of all tested products (2011, in-vitro, data on file)

## Ordering Information

	Size	Ref-No.	Dressings/Box	Suggested HCPCS**
<b>Cutimed® Siltec® Sorbact® Dressing</b>				
	3 x 3 in	7325100	10	A6212
	4 x 4 in	7325116	10	A6213
	4 x 9 in	7325114	10	A6213
	5 x 5 in	7325101	10	A6212
	6 x 6 in	7325102	10	A6212
	7 x 7 in	7325103	5	A6213
	9 x 9 in.	7325104	5	A6213
	Sacrum 7 x 7 in	7325105	5	A6213
	Sacrum 9 x 9 in	7325106	5	A6213
	Oval 3 x 4 in	7325118	10	A6212

## Indications

**Cutimed® Siltec® Sorbact®** is indicated for use in the management of clean, colonized, contaminated or infected, wounds with Moderate to High exudate levels such as venous leg ulcers, pressure ulcers, diabetic foot ulcers, surgical incisions, skin grafts, and donor sites, lacerations or abrasions. It can also be used as an absorbent secondary dressing on deeper wounds.



Sorbact® is a registered trademark of ABIGO Medical AB.  
 Allewyn® is a registered trademark of Smith & Nephew.  
 Mepilex® is a registered trademark of Molnlycke Healthcare.  
 Biataine® is a registered trademark of Coloplast.  
 Please refer to the product label and/or package insert for full instructions on safe use of these products.

**REIMBURSEMENT DISCLAIMER:** The Centers for Medicare & Medicaid Services (CMS) uses Palmetto GBA as the Pricing, Data Analysis, and Coding (PDAC) Contractor. The PDAC Contractor maintains the Product Classification List, located at [https://www4.palmettogba.com/pdac\\_dmecs/](https://www4.palmettogba.com/pdac_dmecs/). The Product Classification List is a searchable database containing products that have received HCPCS coding verification from PDAC and their corresponding HCPCS codes. BSN Medical Inc., however, makes no representations as to the accuracy of the information contained within the PDAC database, nor any representations as to whether its products are reimbursable under any government sponsored healthcare program and/or private-insurance program. It is the provider's sole responsibility, in consultation with the insurer, if necessary, to determine medical necessity, ensure coverage criteria is met, and submit appropriate codes, modifiers, and charges for services rendered.



Cutimed®,  
an Essity brand

-  /BSNWoundCareUSA
-  @BSNWoundCareUSA
-  [bsnmedical.com](http://bsnmedical.com)

BSN Medical Inc., an Essity company  
 5825 Carnegie Blvd., Charlotte, NC 28209-4633  
 Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325  
 To order toll-free: Tel. (+1) 800 537 1063